



PERFECTCOLL™

OPERATING MANUAL

# PC EYE GOLD 24 K

Eye rejuvenated effect





# Contents



24 carat gold of the highest hallmark	4
Recommended cosmetics	6
Design and technical data	10
Application	12
Rejuvenation effects	14
Indications and contraindications	18
Care and maintenance	19

# 24 carat gold of the highest hallmark




24 carat gold used in **PC EYE GOLD 24 K** is, according to the researchers, a very stable material. During friction and heating, negative ions are emitted, which penetrate into the deeper layers of the skin.

Negative ions emitted by **PC EYE GOLD 24 K** capture free positive ions in the human body and thus neutralize them. In this way, they counteract free radicals in cells, restore balance in the body, as well as stimulate ions in the blood, and improve its flow.



Blood circulation and the rate of metabolism are improved; the deeper layers of the skin are stimulated. These processes are the result of the “regenerative properties” of gold. Pure gold ions are also referred to as active gold. Gold ions can be absorbed through the skin with the cosmetic.

With **PC EYE GOLD 24** you will get a rejuvenated skin effect around the eyes, you will brighten dark circles and reduce wrinkles. Your glance will be radiant and youthful.

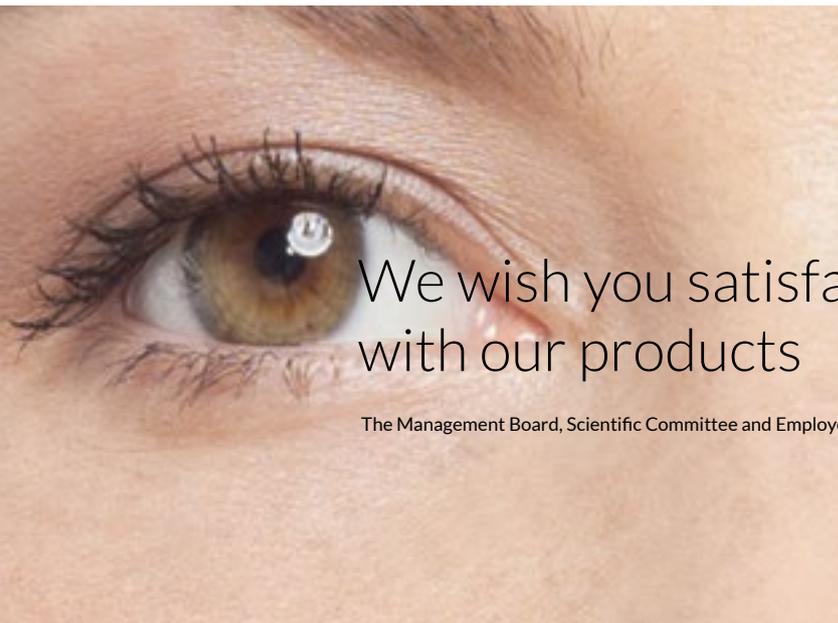

We wish you satisfaction  
with our products

The Management Board, Scientific Committee and Employees of Perfect Coll Sp. z o.o.

# Recommended cosmetics

Use **PC EYE GOLD 24 K** only with the original cosmetics of PERFECT COLL.



1

Before treatment

## PC DETOX EXTRACT FROM NORWEGIAN WILD CORNFLOWER

The cleanest and most modern, all-natural extract of the flowers of Centaurea harvested from organic farming in Scandinavia, enriched with natural enzymes from Organic Kelp (organic algae). It gently cleans and soothes the sensitive skin of face, neck and the whole body. It does not contain parabens, chemical preservatives, and drying skin alcohol.

## SYMBIOSIS OF SCIENCE WITH NATURE

PERFECT COLL cosmetics have unique, innovative formula developed in biotechnology, environmental laboratories in the Polish medical universities.

We are keen on nature and ecology, so the cosmetics with the PC sign are healthy for your skin.



2

During treatment



### PC PREBIOTIC HYALURONIC SERUM

It has innovative and entirely natural prebiotic formula and the most efficient ratio of the various particle sizes of hyaluronic acid, which causes deep hydration of dehydrated or desiccated skin layers. Due to the unique content of prebiotic enzymes, serum is a perfect way for the most demanding skin. It is the only prebiotic product on the market characterized with extremely high absorption.



3 After treatment

### PC PLATINUM COLLAGEN +32°C HIALURON


Poland is famous for producing the purest and best in the world collagen perfectly assimilated by the human body. Our collagen, thanks to the unique technology, has the unique heat-resistance to +32°C. As the only product on the market it may hold fully qualified name of the Platinum, because it contains platinum biopeptide. Platinum facilitates penetration of biologically active substances into the deep layers of the skin, optimizing action of the hyaluronic acid present in the preparation. Excellent composition of three active components in hyaluronic acid enhances the perfect effect after treatment with **PC EYE GOLD 24 K**.



4 After treatment

### PC PREBIOTIC SOOTHING CREAM

Natural and eco-friendly, innovative soothing cream, which due to unique composition has remarkable properties to support the work of damaged skin cells. Its components are responsible for regeneration of even the most delicate skin, and enzymes from papaya fruit provide the effect of light glowing. It is the only product on the market that contains prebiotic enzymes, which provide very high absorbency.

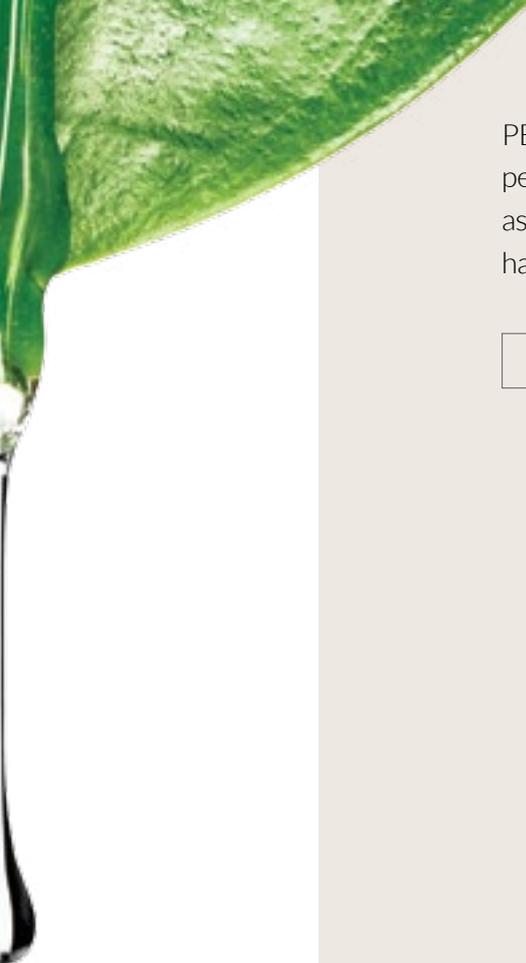

PERFECT COLL cosmetics do not contain parabens, chemical preservatives, petroleum (paraffin), mineral oils, alcohols, and glycols drying out the skin, as well as irritating foaming agent Sodium Laureth Sulfate and other chemical substances harmful to the skin.

Refer to "Perfect Coll Products Catalogue"



# Design and Technical data PC EYE GOLD 24 K





..... Head covered with 24 k gold

Device handle .....



## TECHNICAL DATA

### DEVICE:

Name of the product	PC EYE GOLD 24 K
Dimensions	94.97 x 18.76 x 15.2 mm
Weight	19 g

# Application

# PC EYE GOLD

# 24 K



## SIMPLE STEPS TO USE PC EYE GOLD 24 K:


- Remove make-up.
- Wash the face with **PC DETOX EXTRACT FROM NORWEGIAN WILD CORNFLOWER**.
- Apply the right amount of **PC PREBIOTIC HYALURONIC SERUM** under the eyes using the narrower tip of the spoon.
- Then use the head with 24 carat gold. Slowly move from the temple to the inner corner of the eye, on the eyebrow, with circle movements.
- After the treatment, wash your face with **PC DETOX EXTRACT FROM NORWEGIAN WILD CORNFLOWER**.
- Apply **PC PLATINUM COLLAGEN +32°C HIALURON**, and massage. After treatment, apply **PC PREBIOTIC SOOTHING CREAM** on your face.
- Clean the gold spoon (*Chapter Care and maintenance*).
- Put **PC EYE GOLD 24 K** into the box.



Frequency of treatment:  
Every day



Treatment time:  
10 min.

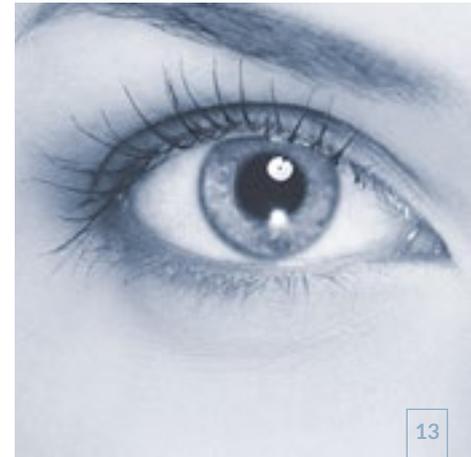

# Effects of rejuvenation

- It prevents from swellings around the eyes, improves blood circulation, removes dark circles under the eyes
- It shallows wrinkles
- It provides anti-ageing and rejuvenating effect
- It has antibacterial properties
- It massages





## **PREVENTION OF PUFFY EYES, IMPROVEMENT OF BLOOD MICROCIRCULATION, REMOVAL OF DARK CURCLES AROUND THE EYES.**

**PC EYE GOLD 24 K** improves blood microcirculation and lymph circulation, which causes that dark circles under the eyes disappear and swelling is decreased. For this reason you quickly achieve the effect of glowing and rejuvenated skin around the eyes.

## **REDUCING WRINKLES**

**PC EYE GOLD 24 K** emits pure gold ions, which stimulate proliferation of collagen, causing quick regeneration of the skin. For this reason, the structure of damaged collagen fibres, as we age, is rebuilt. The result of this process are reduced wrinkles and skin rejuvenation.

## ANTI-AGEING AND REJUVENATING EFFECT

The head of **PC EYE GOLD 24 K** emits clean, negative ions of gold. Ions penetrate deep into the skin and neutralize positive ions in the human body, resulting in elimination of free radicals in cells, stimulation of blood circulation, improvement of metabolism, making the skin to maintain youthful appearance for a very long time.

## ANTIBACTERIAL PROPERTIES

**PC EYE GOLD 24 K** kit has antibacterial and anti-inflammatory properties.

## MASSAGE

Application of **PC EYE GOLD 24 K** to massage improves blood circulation and metabolism in the skin.





# Indications

- Lymphatic stasis.
- Ageing skin.
- Skin with tendency to contamination.
- Swells and dark circles under the eyes.
- Wrinkles around the eyes, and, in particular, “crow’s feet”.



# Contraindications

- Sun-burned skin (in the place of treatment).
- Break in the epidermis (in the place of treatment).
- Bacterial, viral, fungal infection (in the place of treatment).
- Fresh scars, birthmarks and tattoos (in the place of treatments).
- Severe eczema, psoriasis, atopic skin, rosacea (active lesions in the place of treatment).
- Allergy to the components of the cosmetics.
- Surgery provided within the last 6 months (in the place of treatment).
- Treatments including administration of botulinum toxin provided within the last 9 months (e.g. Botox) (in the place of treatment)
- Taking corticosteroids (in the place of treatment).
- Autoimmune diseases (e.g. HIV, lupus, etc.).
- Cancer.
- Tumours, nodules of unknown origin.
- Pregnancy.
- Lactation (breastfeeding).
- Increased temperature.



# Care and maintenance

- After the treatment, remove the preparation remains from the head.
- Wash **PC EYE GOLD 24 K** under running water.
- Wipe with a tissue soaked with antibacterial preparation (e.g. 2% salicylic spirit).
- Put the device into the box.



Keep the device clean and dry.





## Centrum Operacyjno-Rozwojowe Perfect Coll Sp. z o.o.

Pomorski Park Naukowo-Technologiczny (lok. A 210)  
Al. Zwycięstwa 96/98, 81-451 Gdynia



[kontakt@perfect-coll.pl](mailto:kontakt@perfect-coll.pl)



+48 58 767 42 24



[www.perfect-coll.pl](http://www.perfect-coll.pl)

