



PERFECTCOLL™

OPERATING MANUAL

# PC SKIN SCRUB

The effect of glowing and smooth skin

- Cavitation
- Peeling





# Contents



|                                   |    |
|-----------------------------------|----|
| Skin regeneration system          | 4  |
| Recommended cosmetics             | 6  |
| Design and Technical data         | 10 |
| Application                       | 12 |
| Rejuvenation effect               | 16 |
| Indications and contraindications | 18 |
| Care and maintenance              | 19 |

# Skin regeneration system



PERFECT COLL Company presents the high quality portable **PC SKIN SCRUB**.


The device has the function of cavitation peeling. It extremely cleans the skin. Innovative technology allows for removal of keratinized epidermis and restoration of the former condition. In addition, it nutrients the skin, which helps its regeneration process.





**PC SKIN SCRUB** is used mainly for skin cleansing treatments by ultrasonic peeling. As a result of the high-frequency ultrasonic vibration, cell metabolism is accelerated and blood circulation is increased.

Microvibration deeply cleans the stratum corneum of the epidermis. Moreover, it penetrates into the dermis and executes micromassage, which is designed to improve blood circulation.

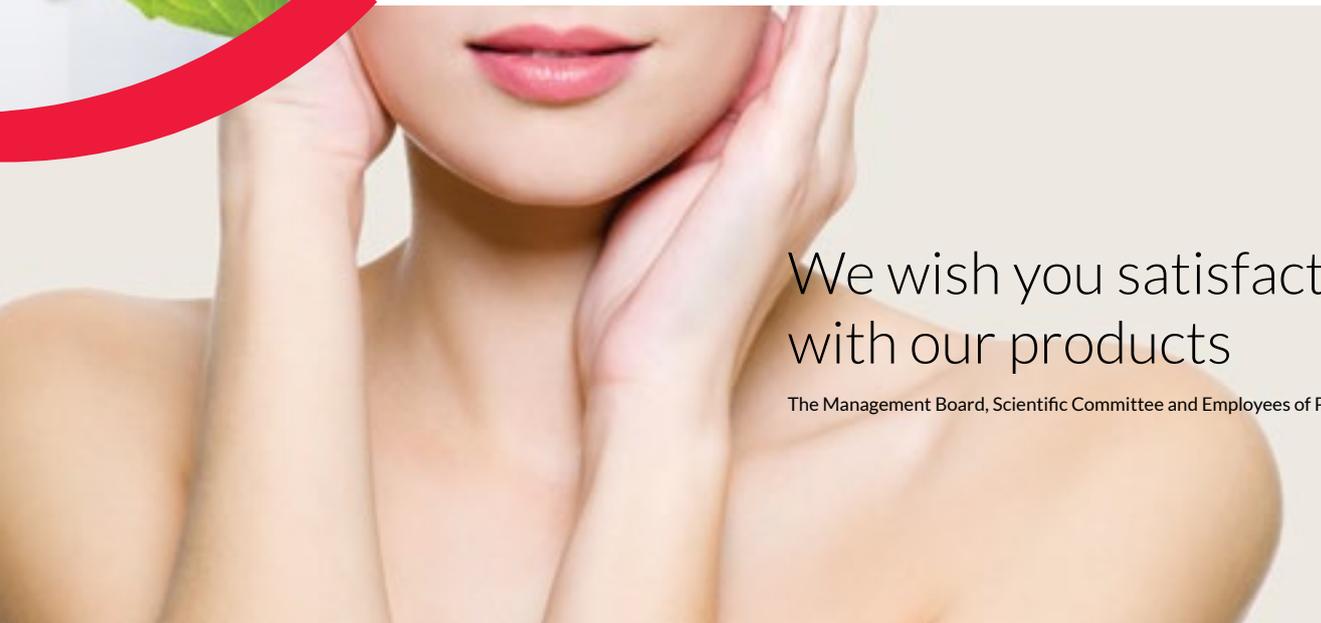

We wish you satisfaction  
with our products

The Management Board, Scientific Committee and Employees of Perfect Coll SP. z o.o.

# Recommended cosmetics

Use **PC SKIN SCRUB** only with the original cosmetics of PERFECT COLL



1

Before treatment

## PC DETOX EXTRACT FROM NORWEGIAN WILD CORNFLOWER

The cleanest and most modern, all-natural extract of the flowers of Centaurea harvested from organic farming in Scandinavia, enriched with natural enzymes from Organic Kelp (organic algae). It gently cleans and soothes the sensitive skin of face, neck and the whole body. It does not contain parabens, chemical preservatives, and drying skin alcohol.

## SYMBIOSIS OF SCIENCE WITH NATURE

PERFECT COLL cosmetics have unique, innovative formula developed in biotechnology, environmental laboratories in the Polish medical universities.

We are keen on nature and ecology, so the cosmetics with the PC sign are healthy for your skin.



2

During treatment



### PC PREBIOTIC HYALURONIC SERUM

It has innovative and entirely natural prebiotic formula and the most efficient ratio of the various particle sizes of hyaluronic acid, which causes deep hydration of dehydrated or desiccated skin layers.

Due to the unique content of prebiotic enzymes, serum is a perfect way for the most demanding skin. It is the only prebiotic product on the market characterized with extremely high absorption.



3 After treatment

### PC PLATINUM COLLAGEN +32°C HIALURON


Poland is famous for producing the purest and best in the world collagen perfectly assimilated by the human body. Our collagen, thanks to the unique technology, has the unique heat-resistance to +32°C. As the only product on the market it may hold fully qualified name of the Platinum, because it contains platinum biopeptide. Platinum facilitates penetration of biologically active substances into the deep layers of the skin, optimizing action of the hyaluronic acid present in the preparation. Excellent composition of three active components in hyaluronic acid enhances the perfect effect after treatment with **PC SKIN SCRUB**.



4 After treatment

### PC PREBIOTIC SOOTHING CREAM

Natural and eco-friendly, innovative soothing cream, which due to unique composition has remarkable properties to support the work of damaged skin cells. Its components are responsible for regeneration of even the most delicate skin, and enzymes from papaya fruit provide the effect of light glowing. It is the only product on the market that contains prebiotic enzymes, which provide very high absorbency.

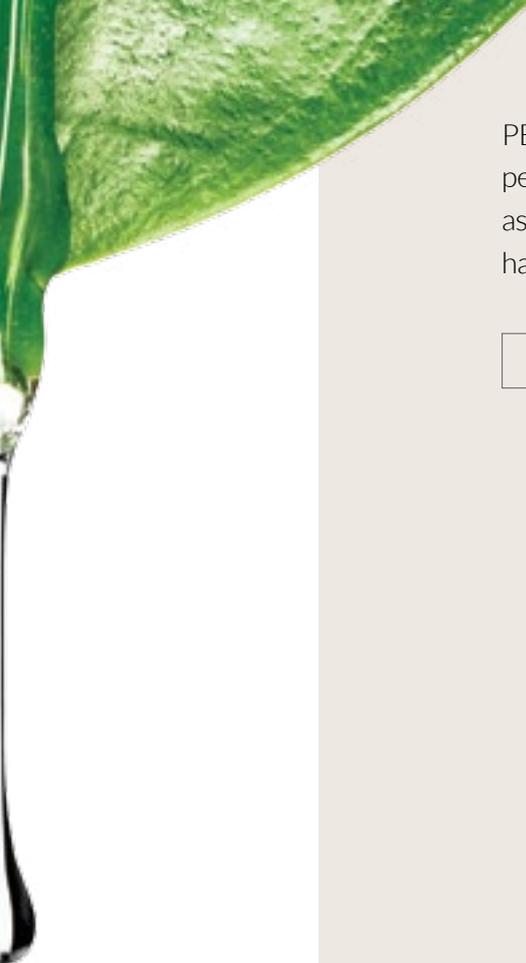

PERFECT COLL cosmetics do not contain parabens, chemical preservatives, petroleum (paraffin), mineral oils, alcohols, and glycols drying out the skin, as well as irritating foaming agent Sodium Laureth Sulfate and other chemical substances harmful to the skin.

Refer to "Perfect Coll Products Catalogue"



# Design and Technical data PC SKIN SCRUB





- ① Spatula
- ② Information panel
- ③ Cleaning indicator
- ④ Nutrition indicator
- ⑤ Cleaning button 1
- ⑥ Cleaning button 2
- ⑦ Nutrition button 1
- ⑧ Nutrition button 2
- ⑨ On / Off

## TECHNICAL DATA

### DEVICE:

|                      |                  |
|----------------------|------------------|
| Name of the Product  | PC SKIN SCRUB    |
| Dimensions           | 170 x 50 x 25 mm |
| Weight               | 87 g             |
| Vibrations frequency | 27 kHz           |

### CHARGER:

|                   |                      |
|-------------------|----------------------|
| Output voltage    | 100-240 V, 50-60 kHz |
| Power consumption | 5V, 1A               |
| Cable length      | 1.15 m               |

### BATTERY:

|                     |               |
|---------------------|---------------|
| Battery             | lithium-ion   |
| Charging time       | 2-6 h         |
| Max. operating time | up to 60 min. |

MADE IN PRC FOR PERFECT COLL



# Application of PC SKIN SCRUB



## CLEANING FUNCTION:

- Thoroughly clean the area subjected to the treatment. It is recommended to use **PC DETOX EXTRACT FROM NORWEGIAN WILD CORNFLOWER**.
- Wet your face with a cotton swab soaked with **PC DETOX EXTRACT FROM NORWEGIAN WILD CORNFLOWER** (do it each time before touching with the spatula).
- Turn on the device by pressing ON/ OFF ⑨.
- Press ⑤ and then ⑥. Cleaning function is activated.
- Put the spatula with concave surface to the skin at an angle of 45°. During the treatment, move the spatula with a slow movement on the skin in strips of approx. 5 to 10 cm, in the direction "from yourself".
- After treatment, apply **PC PLATINUM COLLAGEM +32°C HIALURON PC** and massage.
- At the of the treatment, apply **PC PREBIOTIC SOOTHING CREAM**.



Frequency of treatment:

Once a week  
Problematic skin - twice  
a week



Treatment time:

10-15 minutes




## NUTRITION FUNCTION:

- Thoroughly clean the area subjected to treatment. It is recommended to use **PC DETOX EXTRACT FROM NORWEGIAN WILD CORNFLOWER**.
- Turn on the device by pressing ON/OFF (9).
- Press (7) and then (8). Nutrition function is activated.
- Apply the right amount of **PC PREBIOTIC HYALURONIC SERUM** on the face.
- Put the spatula with concave surface to the skin at an angle of 45°. During the treatment, move the spatula with a slow movement on the skin in strips of approx. 5 to 10 cm, in the direction "from yourself".
- When you feel that your skin is dry, finish the treatment and wash the face with **PC DETOX EXTRACT FROM NORWEGIAN WILD CORNFLOWER**.
- After treatment, apply **PC PLATINUM COLLAGEM +32°C HIALURON** and massage.
- At the end of the treatment, apply **PC PREBIOTIC SOOTHING CREAM**.





# Rejuvenation effects of PC SKIN SCRUB

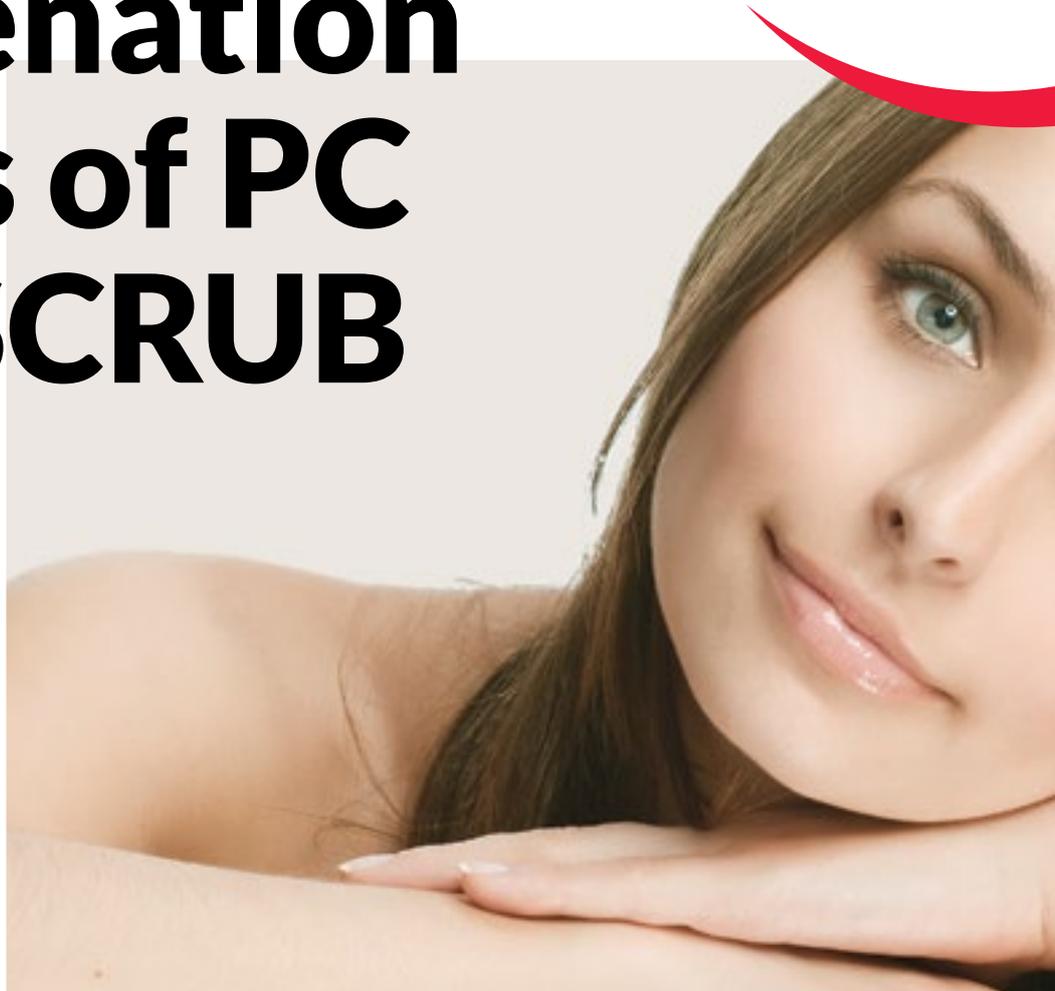



## PC SKIN SCRUB:

- It accelerates metabolism.
- It provides elasticity to the skin.
- It cleans the skin.
- It shallows wrinkles.
- It lightens discolouration.
- It eliminates dead cells.
- It helps in acne treatment (deep cleanses clogged pores).
- It provides active ingredients deep into the skin.
- It improves strength of the walls of the blood vessels.
- It revitalises the skin.



# Indications

- Skin with discolouration.
- Deep wrinkles.
- Lack of firmness and flexibility.
- Dry, grey skin, without glow.
- Contaminated skin.
- Acne skin.
- Acne scars.

# Contraindications

- Sun-burned skin (in the place of treatment)
- Break in the epidermis (in the place of treatment).
- Bacterial, viral, fungal infection (in the place of treatment).
- Fresh scars, birthmarks and tattoos (in the place of treatment).
- Severe eczema, psoriasis, atopic skin, rosacea (active lesions in the place of treatment).
- Allergy to any of the components contained in cosmetics.
- Venectasia and telangiectasia (broken veins on the legs and face) and varicose veins (in the place of treatment).
- Surgery provided within the last 6 months (in the place of treatment).
- Treatments including administration of botulinum toxin provided within the last 9 months (e.g. Botox) (in the place of treatment).
- Fillings with gold thread (in the place of treatment).
- Allergy to metal.
- Metal implants (including teeth and joints, and braces).
- Heart valves, pacemakers (applies to the action of ionopheresis, galvanisation and ultrasound).
- Use of corticosteroids in the place of treatment.



- Taking anticoagulants (blood thinners).
- Autoimmune diseases (e.g. HIV, lupus, etc.).
- Cancer.
- Cardiovascular disease.
- Epilepsy.
- Tumours, nodules of unknown origin.
- Arterial hypertension.
- Tics.
- Cardiac dysrhythmia.
- Pregnancy.
- Lactation (breastfeeding).
- Increased temperature.
- Age below 15 years.



Keep the device away from children.

# Care and maintenance

- Before and after use, keep the device dry and clean.
- Do not use the device close to eyes.
- Keep away from children.
- Disconnect the device when it is not used.
- Do not use with wet hands.
- Remember to disinfect the device.
- After treatment, clean the spatula with a tissue soaked in antibacterial preparation (e.g. 2% salicylic alcohol).



Keep the device clean and dry.





## Centrum Operacyjno-Rozwojowe Perfect Coll Sp. z o.o.

Pomorski Park Naukowo-Technologiczny (lok. A 210)  
Al. Zwycięstwa 96/98, 81-451 Gdynia



[kontakt@perfect-coll.pl](mailto:kontakt@perfect-coll.pl)



+48 58 767 42 24



[www.perfect-coll.pl](http://www.perfect-coll.pl)

